

Dear Client and Eye Care Professional:

If you will be supplying Rx lenses for your Irlen Spectral Filter, this letter is to provide guidance for you and your eye care professional in the selection of your Rx lenses. **The Irlen Institute does not accept responsibility for replacing any client-supplied Rx lenses which are untintable.**

- **Please supply tintable hard resin plastic (CR39) lenses with no UV or anti-reflective glare coating.**
- **High impact resistant lenses, such as Polycarb and Trivex lenses, are not recommended for Irlen Tints.**
 - **Polycarb and Trivex** lenses will generally not take Irlen tints as the Irlen color cannot penetrate the hard coating on these lenses. Polycarb/Trivex lenses may not absorb the color or fade faster. To create the Irlen colors, the lenses need to be in a hot dye and the Polycarb/Trivex lens may crack, making the lens unusable.
- **High impact resistant lenses recommended: Tintable CR-39 industrial thickness 3.0, (tintable hard coat high index 1.60 is acceptable).**
- **Progressive and High-Index Lenses**
 - All **progressive** no-line multi-focal lenses come with the manufacturer's factory scratch coating
 - All **high-index** lenses come with the manufacturer's factory scratch coating and UV400 filtering.
 - These lenses will generally tint, depending on the density/darkness of the tint combination.
- **LensCrafters and Costco** normally provide the Dura Lens with a factory scratch coating that is not tintable. Make sure you request and receive tintable lenses.
- **Grooved Lenses and Rimless Frames, including Rimlon Rimless. Do not send the frame.**
 - Send only lenses. Your eye care professional can mount the lenses in the frame after they have been tinted and returned to you.

For Eye Care Professional: Please identify the type of Rx lenses being sent by checking the appropriate items below and signing the form. **This form must be submitted with the lenses.** If you have questions, please call the Irlen Institute International Headquarters in Long Beach, California, at 1-800-55-IRLEN (47536) or email to **lab@irlen.com**.

To: Eye Care Professional

Client's Name: _____

This form must accompany all Rx lenses sent for tinting. These lenses are:

- | | |
|---------------------------------|--------------------------------------|
| ___ Tintable CR39 lenses | ___ Tintable scratch coating |
| ___ Tintable progressive lenses | ___ No scratch coating |
| ___ Tintable hi-index lenses | ___ No UV coating |
| | ___ No anti-reflection glare coating |

Signature of Eye Care Professional

Date